

NEW!

from Australia's leading supplier of quality magnetic products



magnets4therapy

MAGNETIC ANKLE SUPPORT

4 x 25mm 10,000 gauss Neodymium magnets
in cotton lined Neoprene support



NATURAL PAIN RELIEF DESIGNED FOR ACTIVE WEAR

Lightweight and supportive, strategically placed magnets provide a complete magnetic field for maximum coverage to:

- increase circulation
- accelerate healing
- minimise swelling
- speed recovery



This product may be worn while working, playing sport or resting
DESIGNED IN AUSTRALIA FROM CLINICAL EXPERIENCE

For more information: www.magnets4therapy.com.au or Tel: 1300 733 771



magnets4therapy

MAGNETIC ANKLE SUPPORT

4 x 25mm 10,000 gauss Neodymium magnets in cotton lined Neoprene support

This popular product has been designed with functionality and comfort in mind and it is able to be worn while working, playing sport or resting. The ankle support has been designed for a comfortable fit. Measure around the ankle for a correct fit. For a comfortable fit and effective results the support should not feel too tight nor too loose. If in doubt try a larger size.

PRECAUTIONS

Do not use this product if internal bleeding is suspected or if bleeding externally. The product should not be worn with oils or liniments. The product has been designed to be worn for a maximum of eight (8) hours per day to avoid adaptation process.

PRODUCT CARE

The product is made from supportive neoprene and lined with cotton terry towelling to absorb moisture. Please hand wash in mild soapy water and allow to drip dry.

WARRANTY

The utmost care has been taken in the production of this product, however for your peace of mind, a six (6) month guarantee from date of purchase applies for faulty workmanship or materials. This warranty excludes defects due to normal wear, abuse, accident, neglect, shipping damage or failure to use products in accordance with the instructions.

DISCLAIMER

Individuals with cardiac pacemakers, internally placed insulin pumps, automatic defibrillators or any other type of electro medical device should not use magnetic products without consulting a physician. Do not use magnets near transdermal patch or during pregnancy. Keep magnets away from video, audio media, computer discs, credit cards, and computer or television screens.

LIKE THIS PRODUCT?

Why not try another of our products in the active wear range. **magnets4therapy** also supplies quality magnetic bedding and jewellery. For more information visit our website or contact us:

Tel: 1300 733 771

Web: www.magnets4therapy.com.au

PO Box 99 Beverly Hills NSW 2209 Australia

ABN 77 687 219 205

magnets4therapy is Australia's leading supplier of quality magnetic products